




SPARE PARTS MANUAL

Luggie[®].Chair

FRW044AAUKNA


Luggie Chair

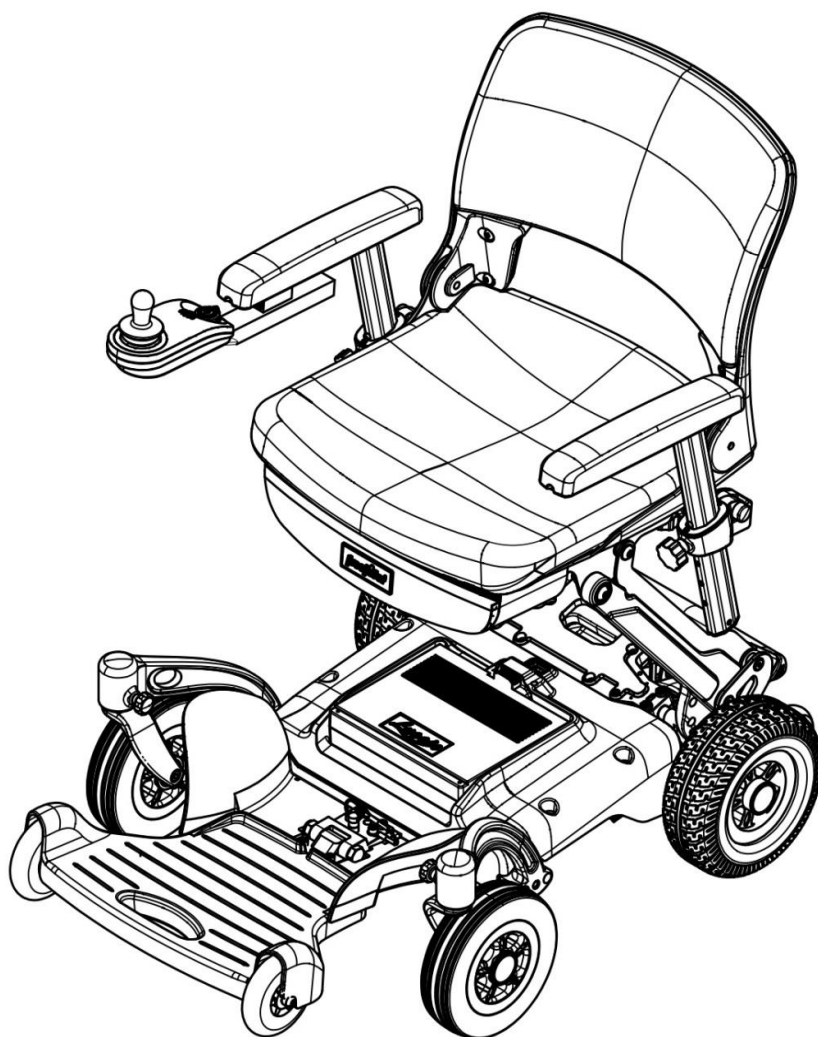

INDEX

1. CHASSIS COVER AND BATTERY ASSEMBLLY	
-----	P.4
2. CHASSIS ASSEMBLY	P.6
3. DRIVE SYSTEM	P.8
4. STEERING	P.10
5. SEAT FOLDING ASSEMBLY	P.12
6. CONSOLE AND WRING SYSTEM	P.14


1. CHASSIS COVER AND BATTERY ASSEMBLY

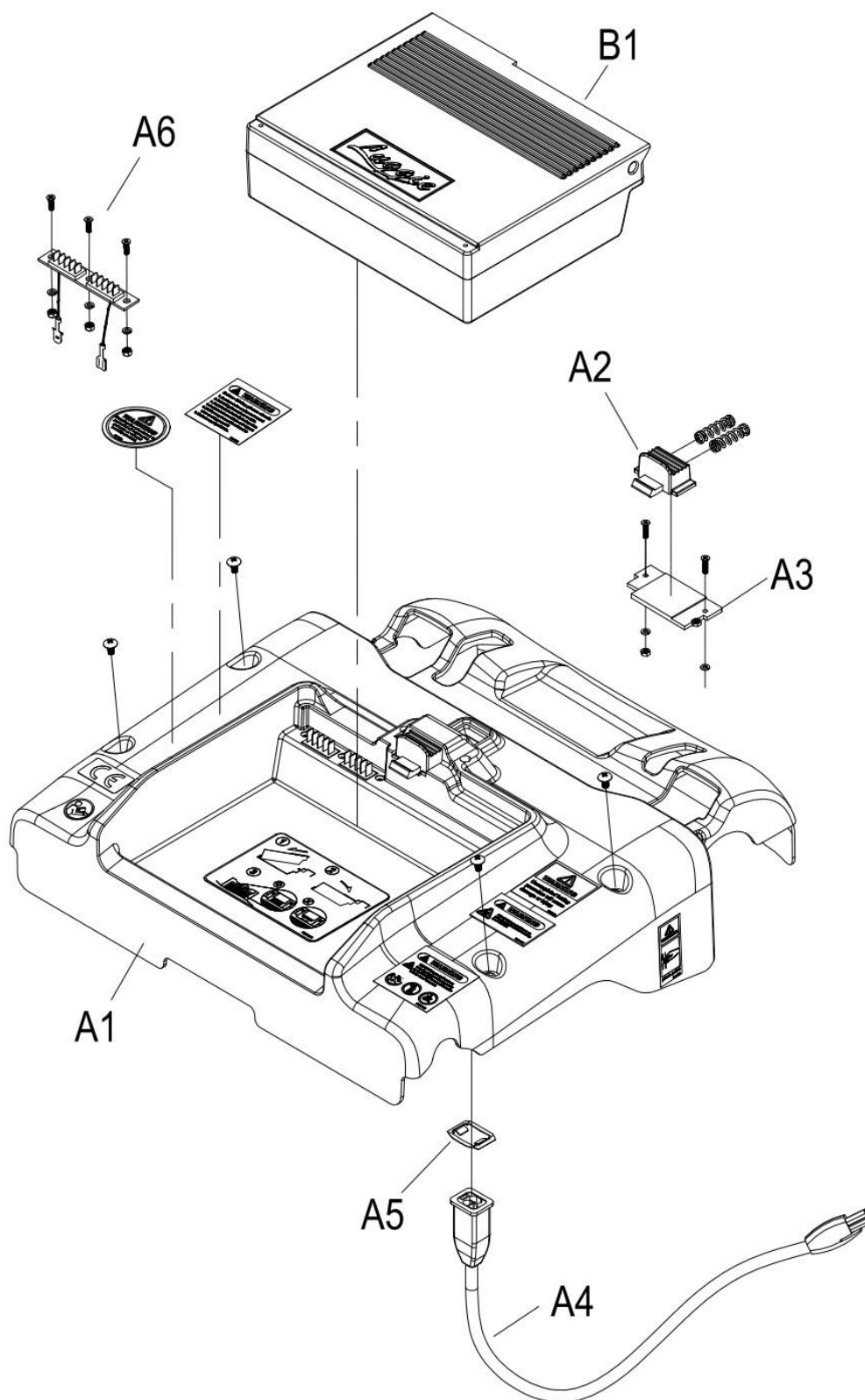

1. CHASSIS COVER AND BATTERY ASSEMBLY

[illegible]


2. CHASSIS ASSEMBLY

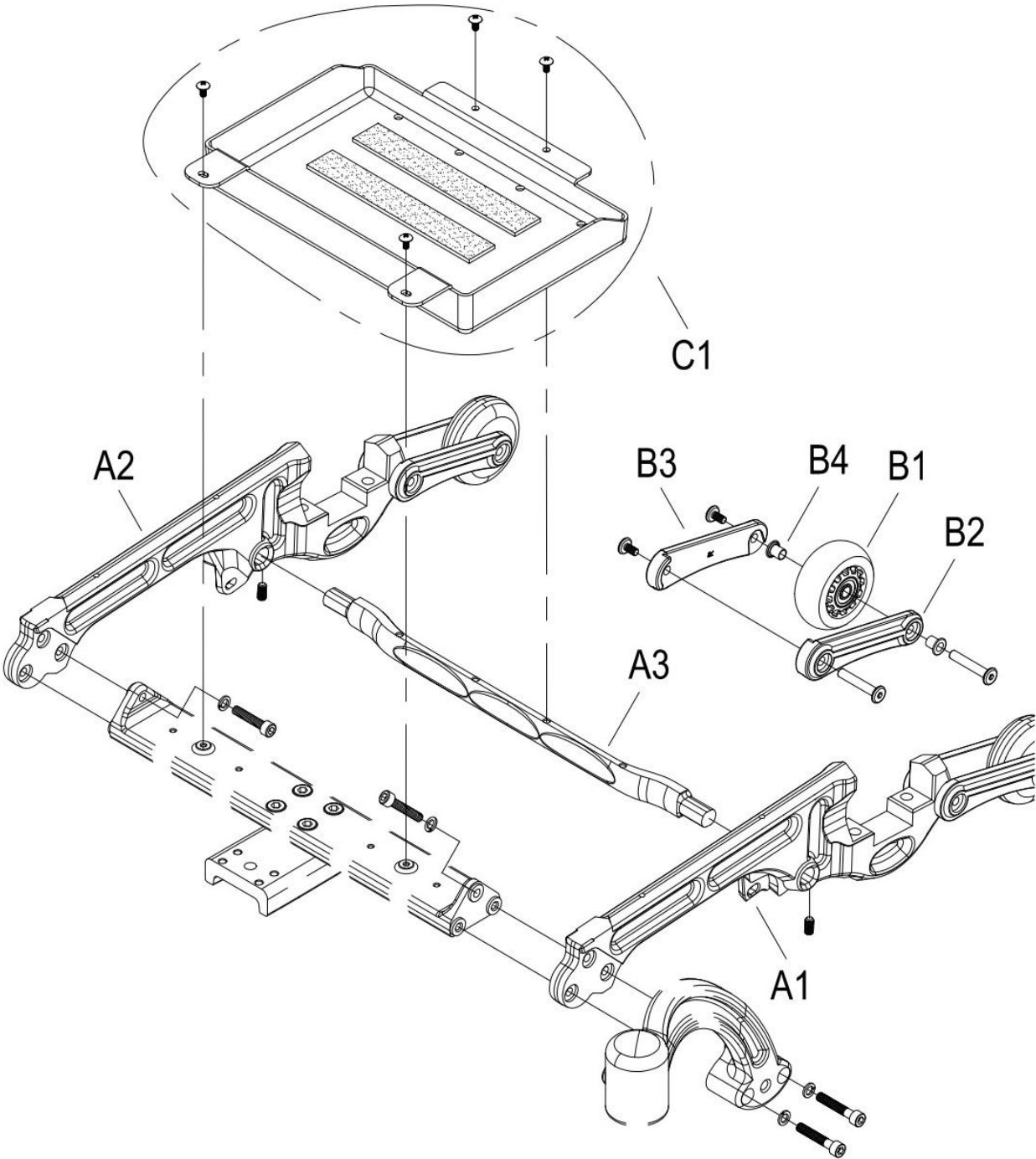

2. CHASSIS ASSEMBLY

[illegible]


3. DRIVE SYSTEM

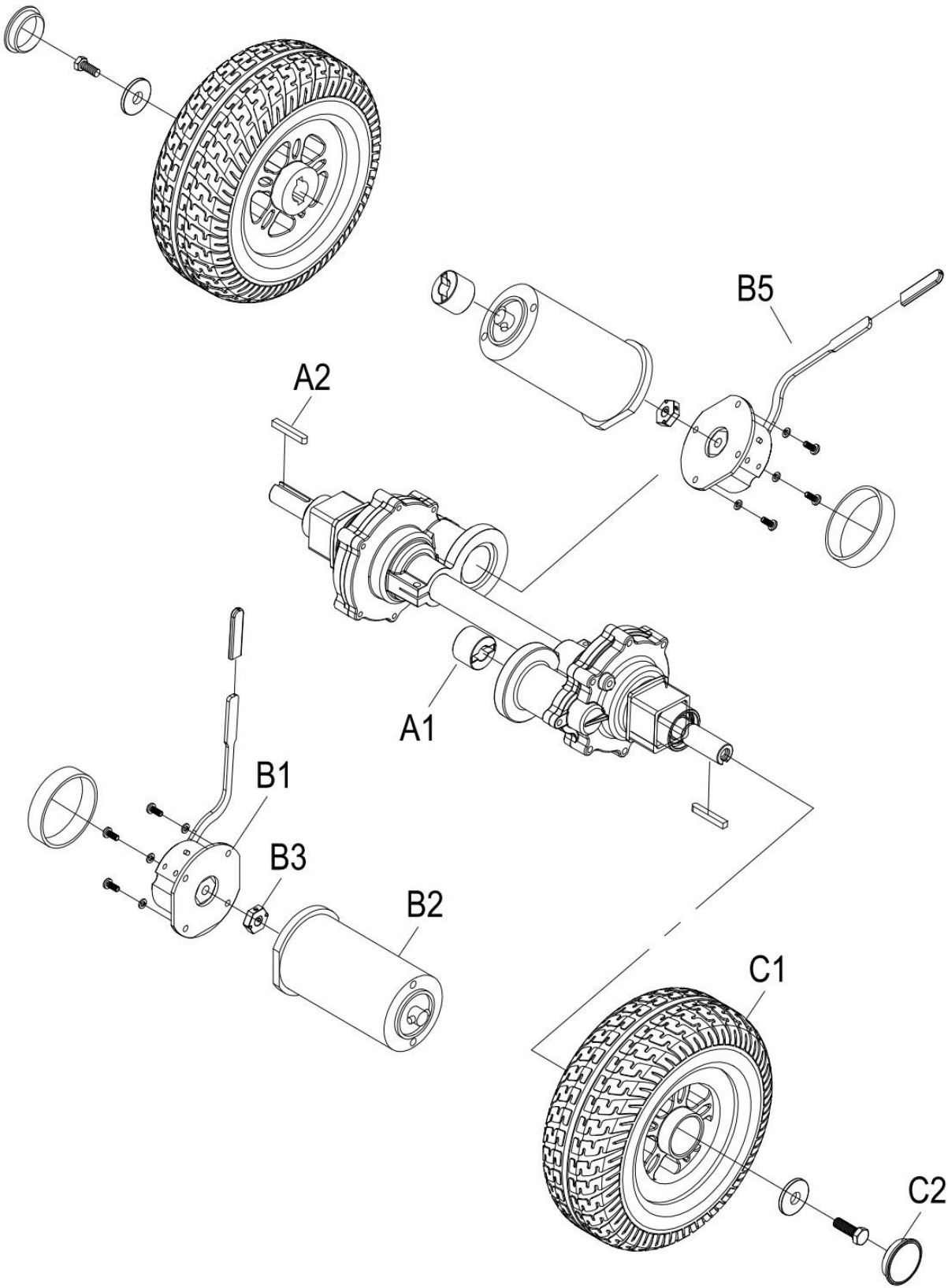

3. DRIVE SYSTEM

[illegible]


4. STEERING

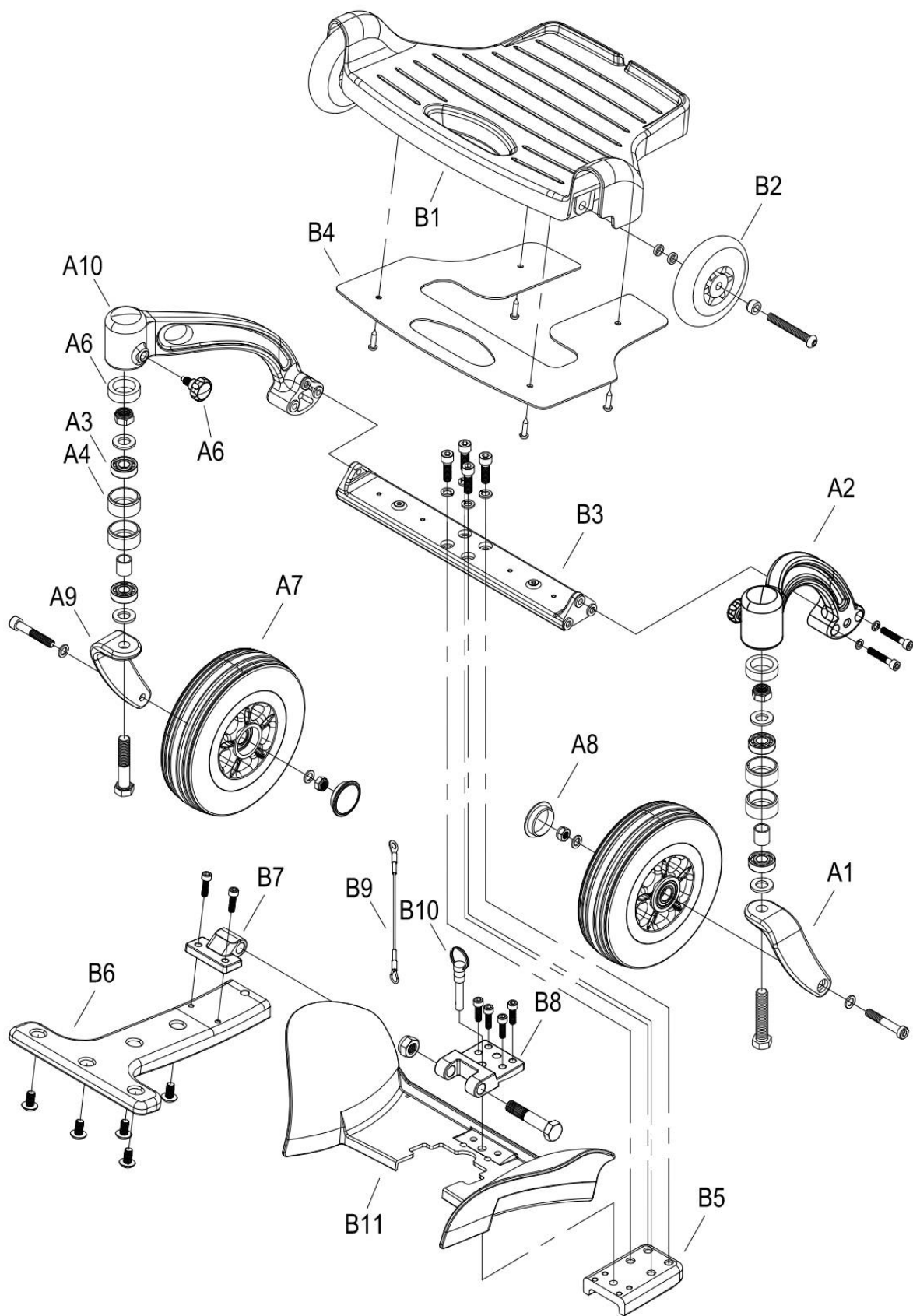

4. STEERING

販售 Selling	項目 ITEM	品號 Part Number	中文品名 Part Name-CN	英文品名 Part Name-EN	數量 QTY	備註 Remark	GBP
	A1 – B11						
	A1 – A6	PA09-B100-L	前叉總成-左	Steering fork assy.(Left)	1		
	A1	PM08-0109-L	前叉-左	Steering fork (Left)	1		
	A2	PM06-1900-L	活動輪支架-左	Front caster arm (Left)	1		
	A3	PM00-0302	軸承-6001 2RS-GR2	Bearing	2		
	A4	PM00-0400	活動輪快拆軸心	Axis	1		
	A5	PP00-0206	拉銷旋鈕	Tghten knob	1	M8×10L	
	A6	PP00-0407	優力膠	PU column	1		
	A7	AC12-1300	前輪總成	Front wheel assy.	1	7"	
	A8	PP91-0002	輪圈蓋	Rim cap	1		
	A3 – A10	PA09-B100-R	前叉總成-右	Steering fork assy.(Right)	1		
	A9	PM08-0109-R	前叉-右	Steering fork (Right)	1		
	A10	PM06-1900-R	活動輪支架-右	Front caster arm(Right)	1		
	B1 – B11	AB08-0201	腳踏板小總成	Footplate assy.	1		
	B1	PP04-1302-D1	腳踏板	Footplate	1		
	B2	PA07-A116	PU彈性輪小總成	PU wheel assy.	2		
	B3	PM06-1001	踏板橫桿	Footplate frame	1		
	B4	PM06-1010	踏板鋁片	Aluminous plate	1		
	B5 – B11	AB08-0202	腳踏板小總成	Footplate assy.	1		
	B5	PM06-1002-F	腳踏板固定架-前	Footplate frame (Front)	1		
	B6	PM06-1002-R	腳踏板固定架-後	Footplate frame (Rear)	1		
	B7	PM90-1905-F	踏板轉折器-前	Footplate hook (Front)	1		
	B8	PM90-1905-R	踏板轉折器-後	Footplate hook (Rear)	1		
	B9	PM00-0222	固定鋼線	Anti-drop wire	1		
	B10	PM00-0211	插銷	Pin	1		
	B11	PP07-0601	擋腳板	Caster shroud	1		


5. SEAT FOLDING ASSEMBLY

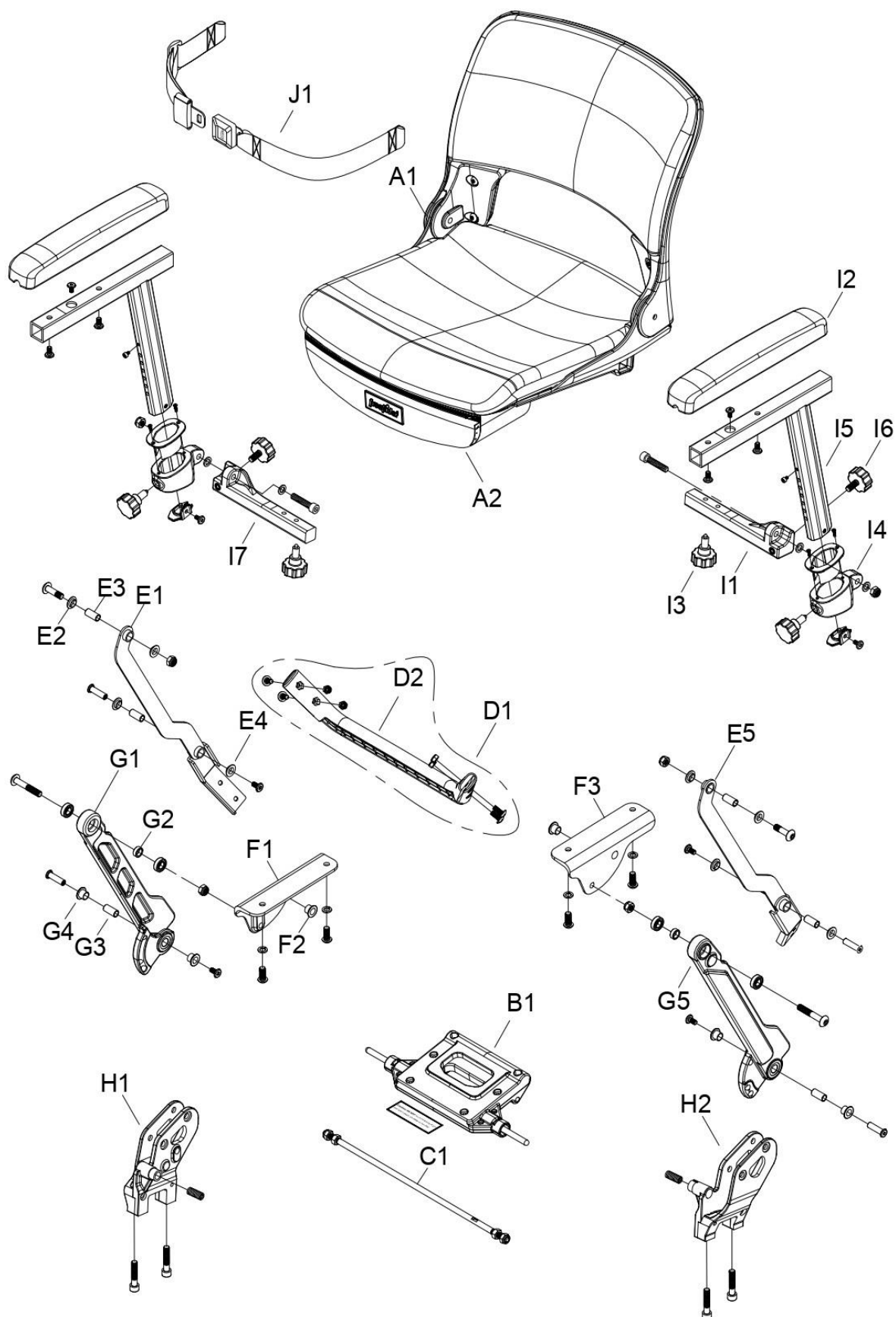

5. SEAT FOLDING ASSEMBLY

販售 Selling	項目 ITEM	品號 Part Number	中文品名 Part Name-CN	英文品名 Part Name-EN	數量 QTY	備註 Remark	GBP
	A1	PA14-A313-001	座椅總成	Seat assy.	1		
	A2	PP90-0513	座椅置物袋	Seat bag	1		
	B1	PA13-A111	座椅固定小總成	Seat folding assy.	1		
	C1	PM09-0314	座椅連桿	Connecting rod	1	M8×P1.25×371L	
	D1	PA07-A420	塑膠後保桿小總成	Rear bumper assy.	1		
	D2	PP07-0501	塑膠後保桿	Rear bumper	1	加長	
	E1－E4	PA15-FA10-R	座椅支撐板小總成-右	Seat support link assy. (Right)	1		
	E1	PM06-0322-R	座椅支撐板-右	Seat support link (Right)	1		
	E2	PP06-0601	T型襯套	Plastic Bushing	2	ψ10×ψ12×4L	
	E3	PM00-0115	套筒	Tube	2	ψ10×ψ7.9×21.5L	
	E4	PP06-0602	T型襯套	Plastic Bushing	2	ψ10×ψ12×10L	
	E2－E5	PA15-FA10-L	座椅支撐板小總成-左	Seat support link assy. (Left)	1		
	E5	PM06-0322-L	座椅支撐板-左	Seat support link (Left)	1		
	F1－F2	PA15-0401-R	椅座承板小總成-右	Seat bracket assy.(Right)	1		
	F1	PM15-0123-R	椅座承板-右	Seat bracket (Right)	1		
	F2	PP06-0602	T型襯套	Plastic Bushing	1	ψ10×ψ12×10L	
	F2－F3	PA15-0401-L	椅座承板小總成-左	Seat bracket assy. (Left)	1		
	F3	PM15-0123-L	椅座承板-左	Seat bracket (Left)	1		
	G1－G5	PA15-FA01-R	座椅連動桿小總成-右	Seat connecting rod assy. (Right)	1		
	G1	PM06-0319-R-A00	座椅連動桿-右	Seat connecting rod (Right)	1		
	G2	PM00-0119	套筒	Tube	1	ψ14.5×ψ10.6×5.5L	
	G3	PM00-0115	套筒	Tube	1	ψ10×ψ7.9×21.5L	
	G4	PP06-0602	T型襯套	Plastic Bushing	2	ψ10×ψ12×10L	
	G2－G5	PA15-FA01-L	座椅連動桿小總成-左	Seat connecting rod assy. (Left)	1		
	G5	PM06-0319-L-A00	座椅連動桿-左	Seat connecting rod (Left)	1		
	H1	PA07-0022-R	座椅托架小總成-右	Support frame bracket assy. (Right)	1		
	H2	PA07-0022-L	座椅托架小總成-左	Support frame bracket assy. (Left)	1		
	I1－I6	AD01-FC01-L	座椅扶手總成-左	Armrest assy.(Left)	1		
	I1	PM14-0216-L	扶手底座-左	Armrest base(Left)	1		
	I2	PP14-0106	扶手墊	Armrest pad	1		
	I3	PP00-0202	拉銷旋鈕	Tighten knob	2	M10×15L	
	I4	PM14-0218-1	扶手轉軸	Armrest hinge joint	1		
	I5	PM14-0402	扶手擠型管	Armrest tube	1		
	I6	PP00-0209	梅花旋鈕	Knob	1		
	I2－I7	AD01-FC01-R	座椅扶手總成-右	Armrest assy.(Right)	1		
	I7	PM14-0216-R	扶手底座-右	Armrest base(Right)	1		
	J1	PP90-0109	安全帶	Seat belt	1		

6. CONSOLE AND WIRING SYSTEM

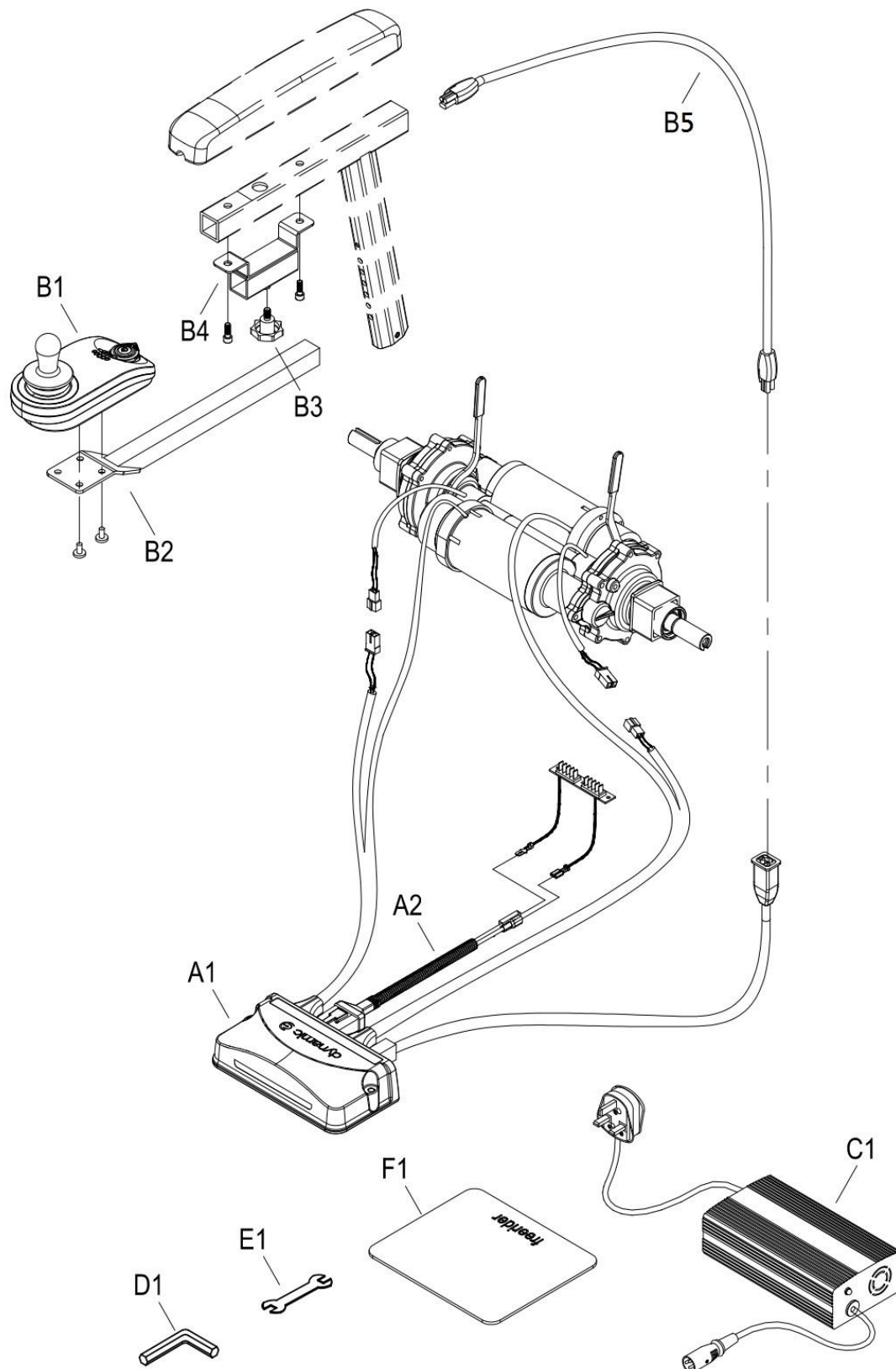

6. CONSOLE AND WIRING SYSTEM

[illegible]