

freerider[®]

SPARE PARTS MANUAL

Luggie[®]

FRL053ASUKNB


FR-L05 Luggie Super Plus



INDEX

1. HANDLEBARASSEMBLY	P.4
2. CHASSIS COVER AND BATTERY ASSEMBLY	P.6
3. STEERING	P.8
4. CHASSIS ASSEMBLY	P.10
5. DRIVE SYSTEM	P.12
6. SEAT FOLDING ASSEMBLY	P.14
7. CONSOLE AND WIRING SYSTEM	P.16


1. HANDLEBAR ASSEMBLY

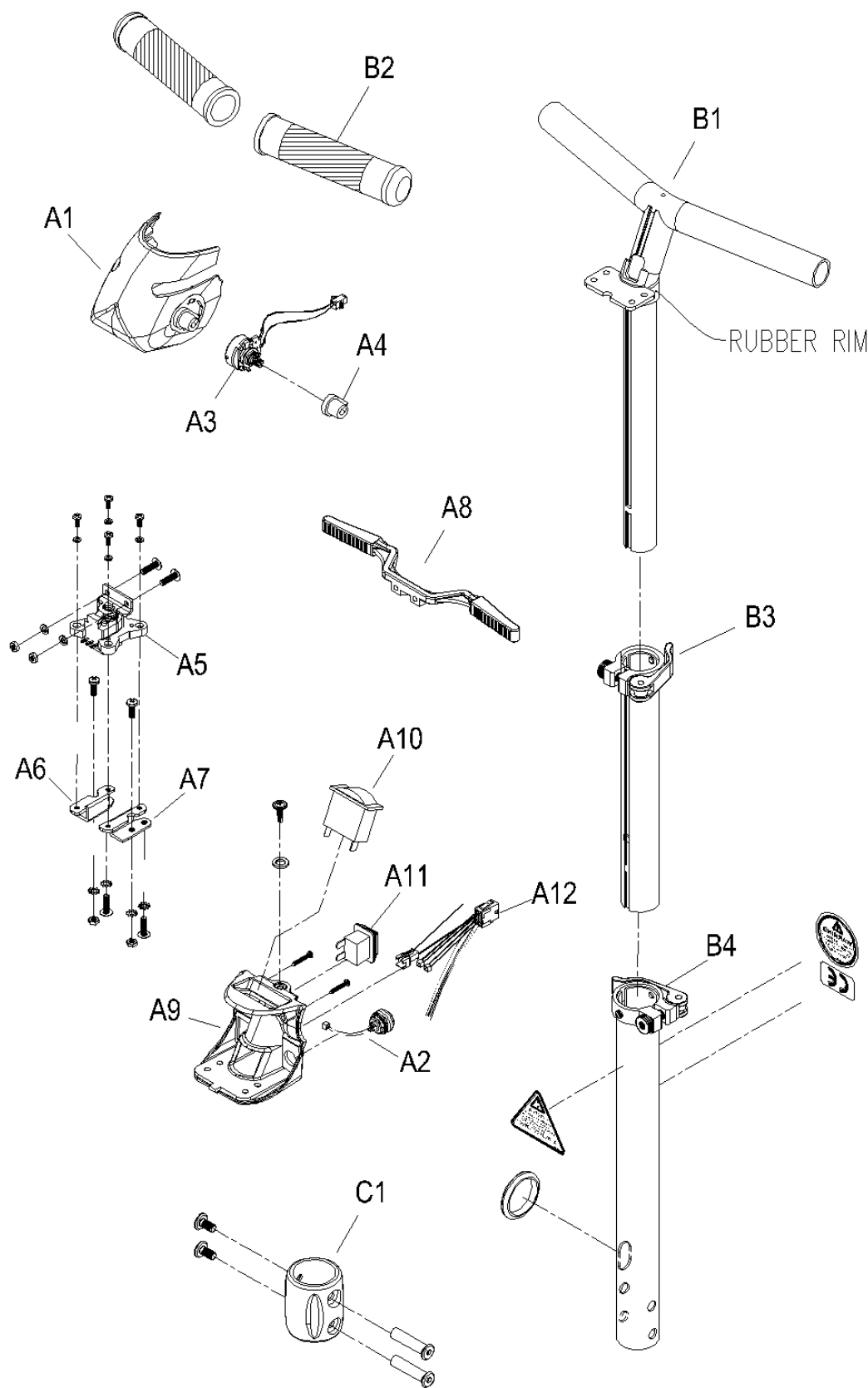

1. HANDLEBAR ASSEMBLY

[illegible]


2. CHASSIS COVER AND BATTERY ASSEMBLY

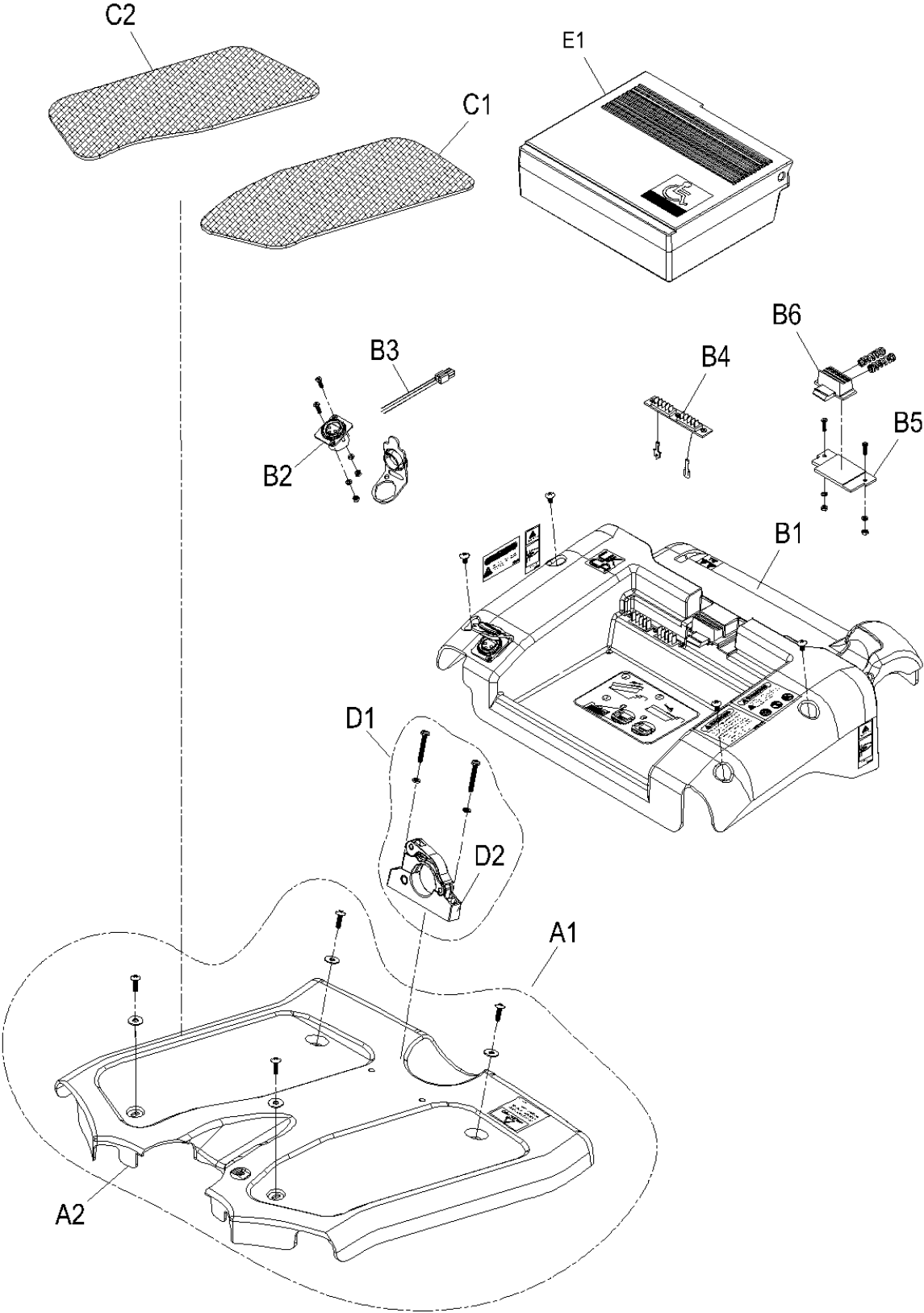

2. CHASSIS COVER AND BATTERY ASSEMBLY

販售 Selling	項目 ITEM	品號 Part Number	中文品名 Part Name-CN	英文品名 Part Name-EN	數量 QTY	備註 Remark	GBP
	A1－E1						
	A1	AA04-4200-◎	前車殼總成	Front chassis cover assy.	1	◎= BS：Ocean Blue、 BG：Light Green、 GG：Mineral Grey	
	A2	PP04-0150-◎	前車殼	Front chassis cover	1	◎= BS：Ocean Blue、 BG：Light Green、 GG：Mineral Grey	
	B1－B6	AA05-2800-◎	後車殼總成	Rear chassis cover assy.	1	◎= BS：Ocean Blue、 BG：Light Green、 GG：Mineral Grey	
	B1	PP05-0160-◎	後車殼	Rear chassis cover	1	◎= BS：Ocean Blue、 BG：Light Green、 GG：Mineral Grey	
	B2－B3	PAE1-0410-NP-A00	充電座小總成	Charge socket assy.	1		
	B2	PKE1-0905-A00	充電座(無PUSH)	Charge socket	1	05M207N-3N2	
	B3	PKE1-0231	4P充電線	Charging cable	1	3條圓Pin	
	B4	PAE1-1400	電池插座小總成	Battery connector assy.	1		
	B5－B6	PAE4-1000	電池前卡鉗小總成	Battery locker assy.	1		
	B5	PP05-0603	電池前卡鉗固定板	Plastic plate	1		
	B6	PP05-0602	電池前卡鉗	Sliding caliper	1		
	C1	PP04-0412-L	止滑踏墊-左	Carpet(Left)	1		
	C2	PP04-0412-R	止滑踏墊-右	Carpet(Right)	1		
	D1	PA07-0010	T把固定座小總成	Tiller clamp assy.	1		
	D2	PM02-1001	T把固定座	Tiller clamp	1		
	E1	PA04-A702-B	電池盒小總成	10.5ah Lithium battery	1	10.5Ah鋰電	


3. STEERING

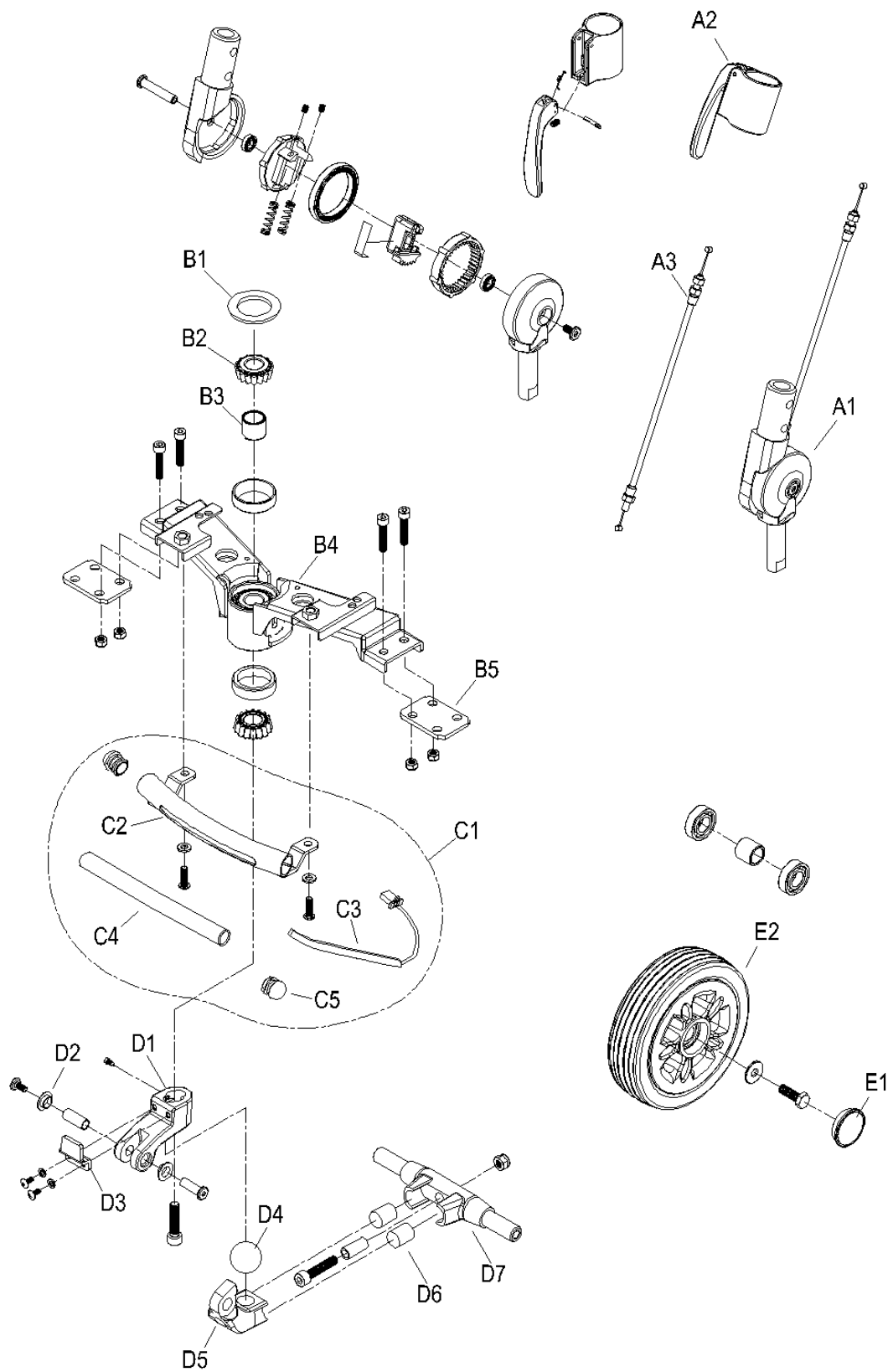

3. STEERING

販售 Selling	項目 ITEM	品號 Part Number	中文品名 Part Name-CN	英文品名 Part Name-EN	數量 QTY	備註 Remark	GBP
	A1－E2						
	A1－A3	AC05-FAXAA-IG	角度調整器總成	Handlebar joint base assy.	1		
	A1	PA10-A104-IG	角度調整器小總成	Handlebar joint base assy.	1	W/reverse cable	
	A2	PA01-A100	板機座小總成	Tiller trigger assy.	1		
	A3	PM19-0306	Y線	Reverse cable	1		
	B1－B5	PA06-B702-01	首管固定座小總成	Front axle assy.	1		
	B1	PP06-0204	首管防塵墊片	Rubber spacer	1	ψ40×ψ26×3mm	
	B2	PM00-0310	軸承	Bearing	2	30203	
	B3	PM00-0120	套管	Bushing	1	do22×di18×16.8l	
	B4	PM06-1212	首管固定座	Front axle	1	含加工軸承套×2	
	B5	PM08-1008	首管延長鐵片	Front axle	2		
	C1	PA06-A422	提把小總成	Front bumper assy.	1		
	C2	PM06-0219	提把	Front bumper	1		
	C3	PKE3-1005	LED防水軟燈條	LED Daylight	1	100mm	
	C4	PP06-0108	矽膠軟管	Hose	1	150mm	
	C5	PP91-0701	管塞	Pipe plug	2		
	D1－D7	PA09-0200	前避震小總成	Shock absorber assy.	1		
	D1－D3	PA09-0202	轉軸上座小總成	Shock absorber assy.	1		
	D1	PM09-0551-GD	轉軸上座	Shock absorber	1	Up	
	D2	PP06-0601	T型襯套	Plastic bushing	2	ψ10×ψ12×4L	
	D3	PM06-1211-1	轉向限制塊	Stopper	1		
	D4－D7	PA09-0203	轉軸小總成	Shock absorber assy.	1		
	D4	PP00-0408	橡膠圓球	Rubber ball	1		
	D5	PM09-0550-GD	轉軸	Shock absorber	1	Down	
	D6	PP00-0409	優力膠圓柱	PU column	2	ψ16×20L	
	D7	PM09-0513-GD	減震軸心	Axle	1		
	E1－E2	AC07-FAAXBB	前輪總成	Front wheel assy.	2	6"	
	E1	PP91-0002	輪圈蓋	Rim cap	1		
	E2	AC07-FAAXBA	前輪	Front wheel	1	LS-W255	


4. CHASSIS ASSEMBLY

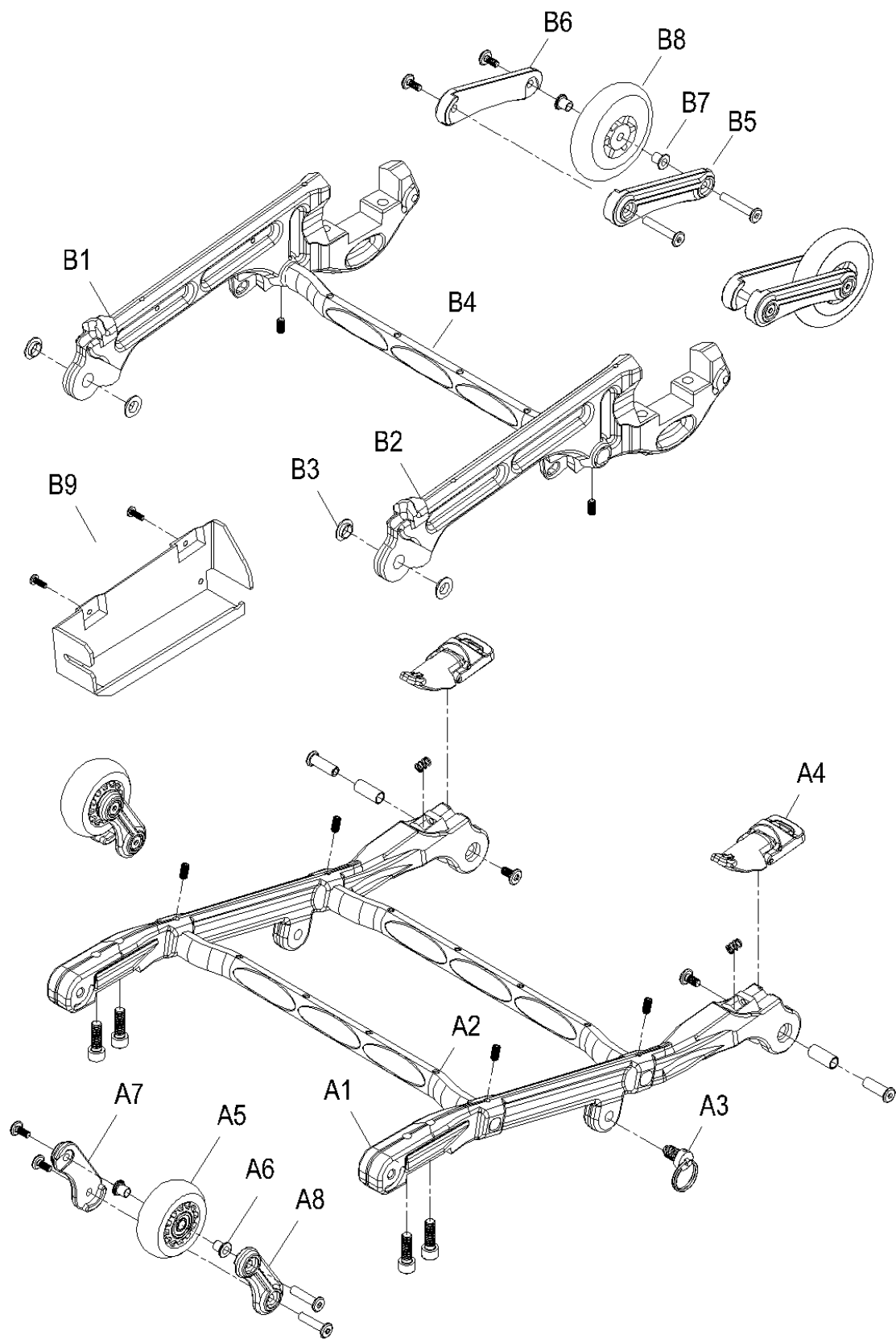

4. CHASSIS ASSEMBLY

[illegible]


5. DRIVE SYSTEM

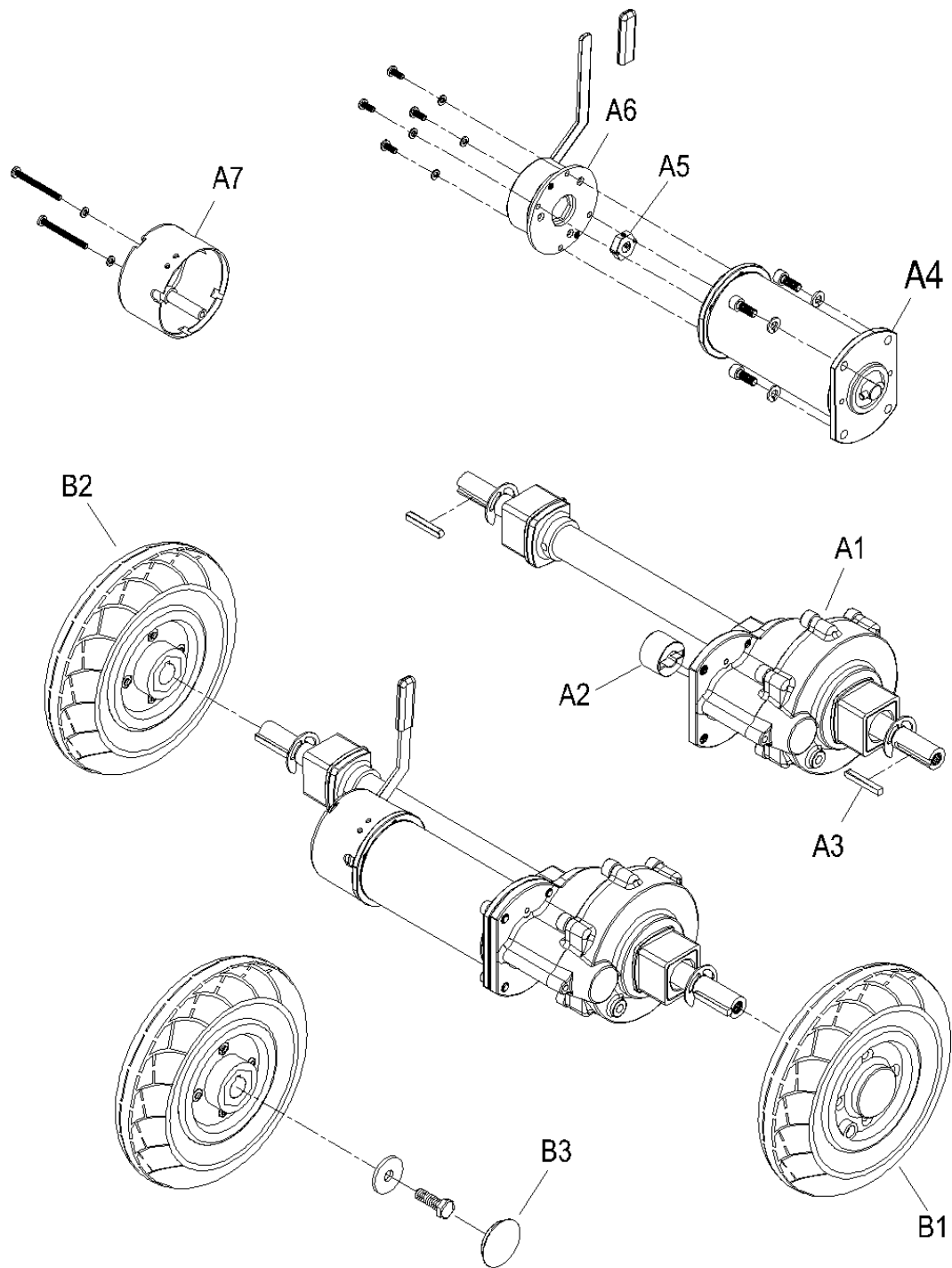

5. DRIVE SYSTEM

[illegible]


6. SEAT FOLDING ASSEMBLY

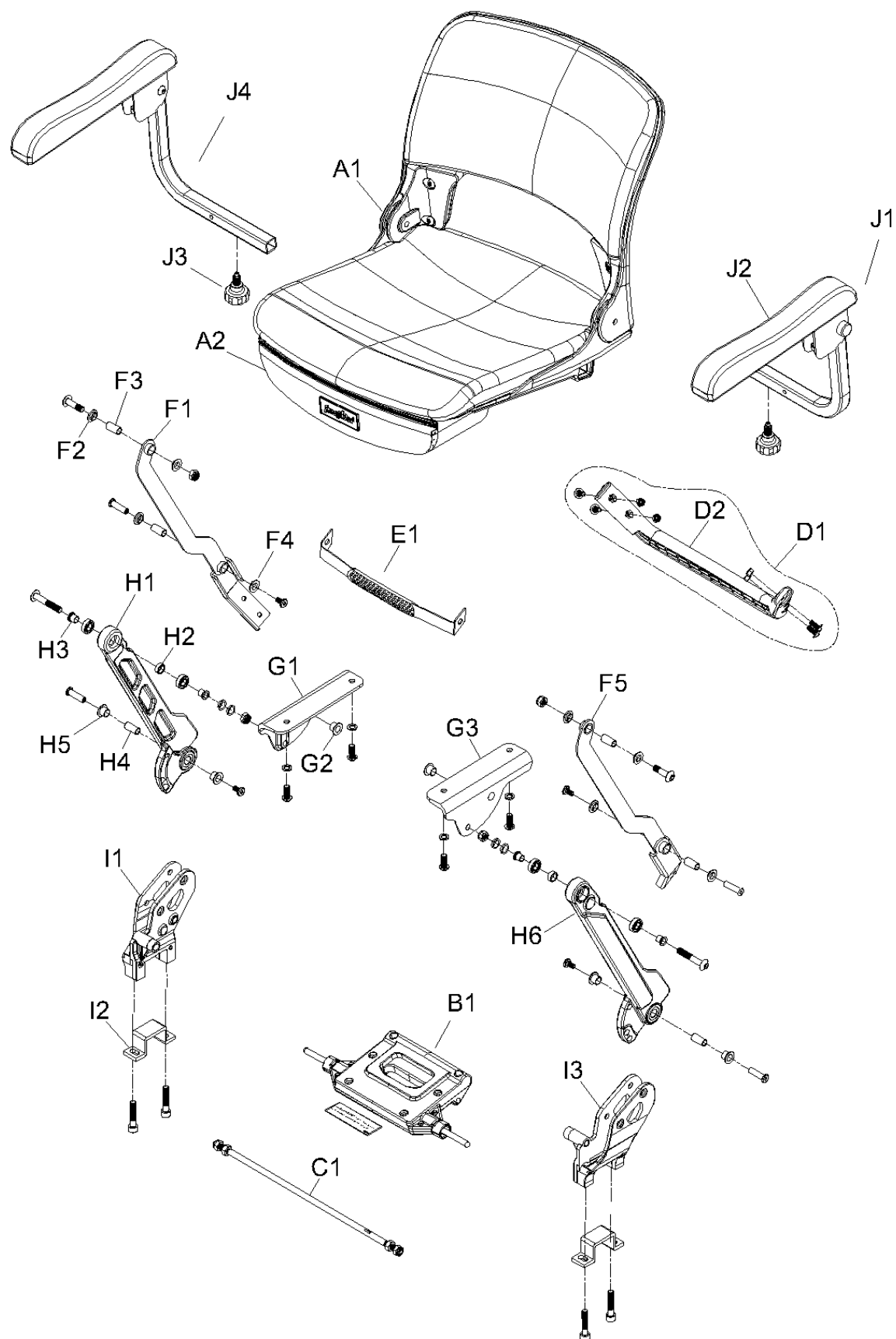

6. SEAT FOLDING ASSEMBLY

販售 Selling	項目 ITEM	品號 Part Number	中文品名 Part Name-CN	英文品名 Part Name-EN	數量 QTY	備註 Remark	GBP
	A1－J4						
	A1	PA14-A313-001	座椅總成	Seat assy.	1		
	A2	PP90-0513	座椅置物袋	Seat bag	1		
	B1	PA13-A111	座椅固定小總成	Seat folding assy.	1		
	C1	PM09-0314	座椅連桿	Connecting rod	1	M8×P1.25×371L	
	D1	PA07-A420	塑膠後保桿小總成	Rear bumper assy.	1		
	D2	PP07-0501	塑膠後保桿	Rear bumper	1	加長	
	E1	PP90-0111	提帶	Carry belt	1		
	F1－F4	PA15-FA10-R	座椅支撐板小總成-右	Seat support link assy. (Right)	1		
	F1	PM06-0322-R	座椅支撐板-右	Seat support link (Right)	1		
	F2	PP06-0601	T型襯套	Plastic bushing	2	ψ10×ψ12×4L	
	F3	PM00-0115	套筒	Tube	2	ψ10×ψ7.9×21.5L	
	F4	PP06-0602	T型襯套	Plastic bushing	2	ψ10×ψ12×10L	
	F2－F5	PA15-FA10-L	座椅支撐板小總成-左	Seat support link assy. (Left)	1		
	F5	PM06-0322-L	座椅支撐板-左	Seat support link (Left)	1		
	G1－G2	PA15-0400-R	椅座承板小總成-右	Seat bracket assy.(Right)	1		
	G1	PM15-0120-R	椅座承板-右	Seat bracket (Right)	1		
	G2	PP06-0602	T型襯套	Plastic bushing	1	ψ10×ψ12×10L	
	G2－G3	PA15-0400-L	椅座承板小總成-左	Seat bracket assy. (Left)	1		
	G3	PM15-0120-L	椅座承板-左	Seat bracket (Left)	1		
	H1－H5	PA15-FA00-R	座椅連動桿小總成-右	Seat connecting rod assy. (Right)	1		
	H1	PM06-0321-R-A00	座椅連動桿-右	Seat connecting rod (Right)	1		
	H2	PM00-0119	套筒	Tube	1	ψ14.5×ψ10.6×5.5L	
	H3	PM00-0118	T套管	Bushing	2	ψ10×ψ8×10L	
	H4	PM00-0115	套筒	Tube	1	ψ10×ψ7.9×21.5L	
	H5	PP06-0602	T型襯套	Plastic bushing	2	ψ10×ψ12×10L	
	H2－H6	PA15-FA00-L	座椅連動桿小總成-左	Seat connecting rod assy. (Left)	1		
	H6	PM06-0321-L-A00	座椅連動桿-左	Seat connecting rod (Left)	1		
	I1－I2	PA07-0021-R	座椅托架小總成-右	Support frame bracket assy. (Right)	1		
	I1	PM15-0307-R	座椅托架-右	Support frame bracket (Right)	1		
	I2	PM07-0302	差速器墊高片	Transaxle plate	1		
	I2－I3	PA07-0021-L	座椅托架小總成-左	Support frame bracket assy. (Left)	1		
	I3	PM15-0307-L	座椅托架-左	Support frame bracket (Left)	1		
	J1	AD01-FB00-L	座椅扶手總成-左	Armrest assy.(Left)	1		
	J2	PP14-0104	扶手墊	Armrest pad	2	橢圓型	
	J3	PP00-0205	拉鎖旋鈕	Tighten knob	2	M10×15L	
	J4	AD01-FB00-R	座椅扶手總成-右	Armrest assy.(Right)	1		

7.CONSOLE AND WIRING SYSTEM

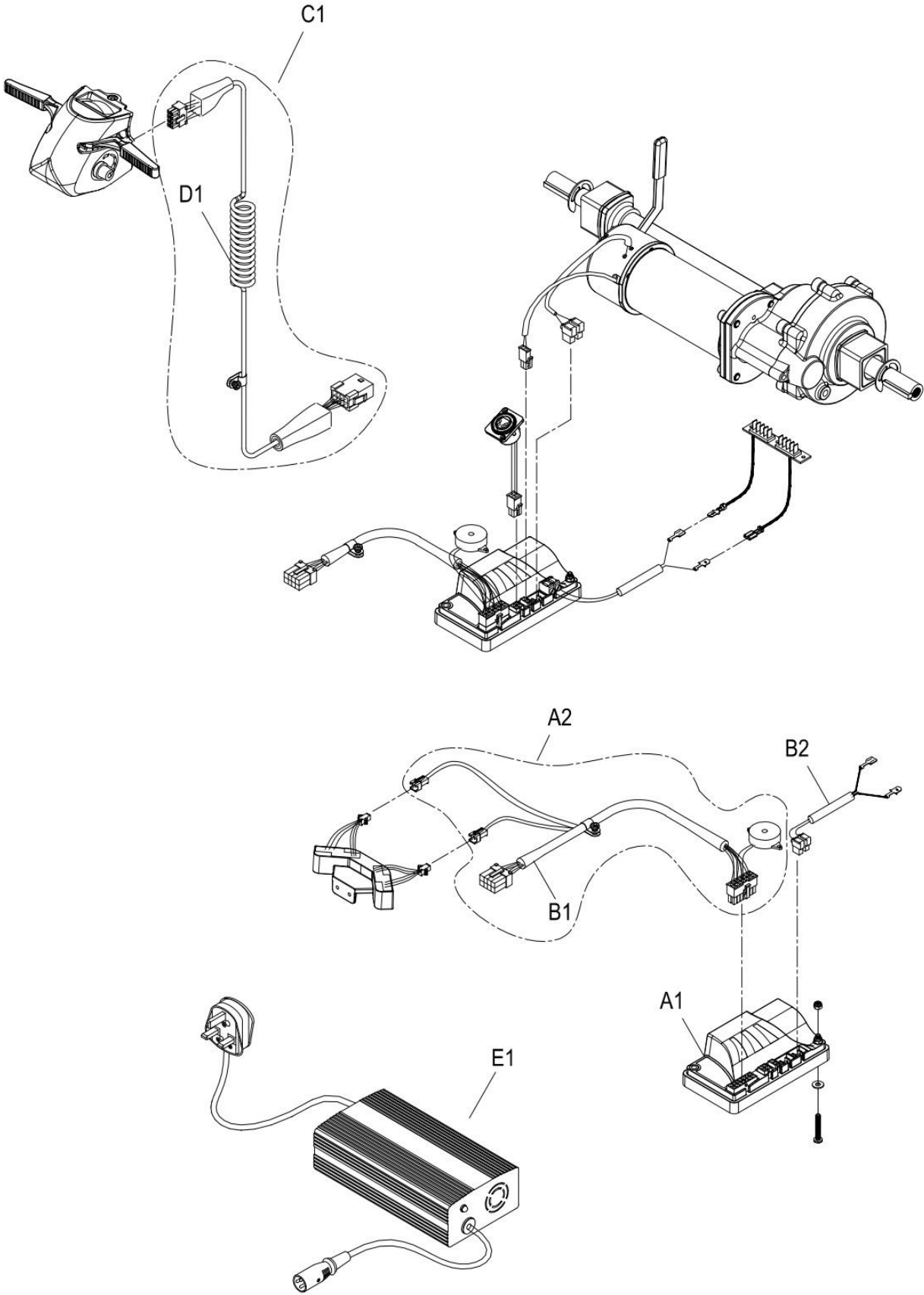

7.CONSOLE AND WIRING SYSTEM

[illegible]